

A16 Automated External Defibrillator



A16

HeartOn AED

**ANYTIME, ANYWHERE,
ANYONE**

Mediana's HeartOn AED helps to save people's lives when Sudden Cardiac Arrest occurs.



CONFIGURATION

1 Connecting Socket
Defibrillator pads connector socket. The pads are always ready for use.

3 Speaker
Acoustic indicator for real time guidance.

5 Power Button
The device will be activated.

7 Pad Connect icon
Guidance on connecting pads.

9 Analysing ECG Icon
Guidance not to touch patient during ECG analysis.

11 CPR Icon
Guidance of the CPR administration with metronome after electric shock.

2 Language Button
User can select the desired language among 3 pre-installed languages.


4 Adult/Pediatric Mode Switch Button
Easily switch from adult to pediatric mode without changing pads.

6 Voice Recording
On-site recording function.

8 Pads Attachment Icon
Guidance on where to attach the pads.

10 Shock Button (Semi-Auto Only)
Flashing button indicates "Ready for shock delivery". Push the button to deliver shock.

12 Status Indicator
AED status, Battery level status, Temperature status, Pads status.

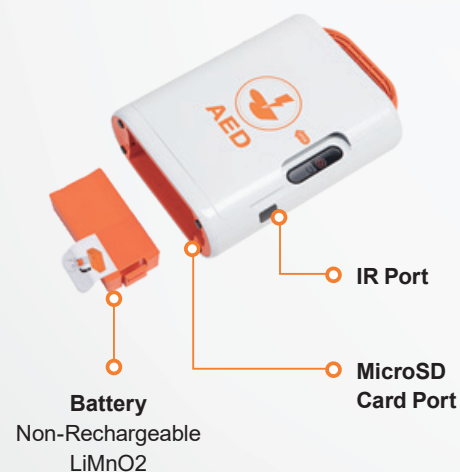

CPR Feedback

FEATURES & FUNCTIONS



1. Reduction of the defibrillation time

- In order to shorten the time before administering the shock, ECG analysis & charging are performed simultaneously.

»» Before



»» After



2. Selection among 3 pre-installed languages

- Ideal for locations with many foreign visitors or countries with several official languages.



3. Pads Quality Check

- Status of pads can be checked on the status LCD: connection problems, expiration date of pads, used pads, damaged pads, or dried gel. The pads can be changed if a problem is indicated.



4. Pacer Pulse Detection & Rejection

- Accurate electrocardiogram analysis is possible after pacing pulses are removed when the AED is applied to a patient with pacemaker.

HOW TO USE



1
Open the cover & press the power-on button



2
Attach the pads






3
Simultaneous ECG analysis & charging



4
Press the shock button (Semi-Auto)

AED EVENT REVIEW SOFTWARE

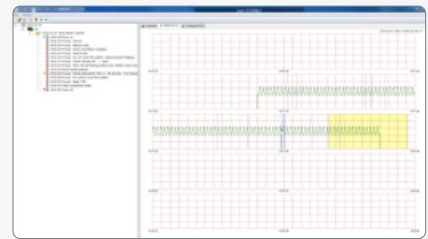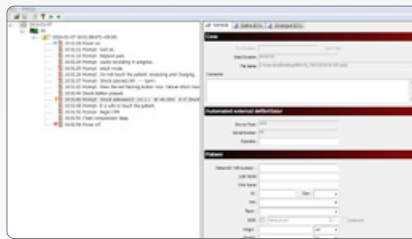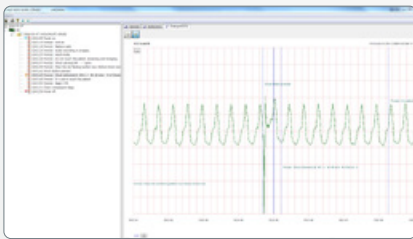

Mediana Seoul Office

53 Donggwang-ro, Seocho-gu, Seoul,
South Korea, 06561

Tel : +82 2 542 3375 Fax : +82 2 542 7447

Mediana Head Office / Factory / R&D

132 Donghwagongdan-ro, Munmak-eup, Wonju-si,
Gangwon-do, South Korea, 26365

Tel : +82 33 742 5400 Fax : +82 33 742 5418

www.mediana.co.kr/eng

